

WHITE SMILE CENTRAL TEETH WHITENING PEN

DIRECTIONS: Remove cap and twist handle several times until a ball of gel appears on brush. Smile wide and spread gel evenly on the front of each visible tooth. Twist handle again to obtain additional gel as needed till thin coat of gel covers all visible teeth. After 60 seconds you may relax your smile. Gel will dissolve and you may swallow. Do not eat or drink anything except water for 60 minutes. Use twice daily until desired results are achieved. Follow up with an application of Minerals™ Enamel Booster.

FOR BEST RESULTS: use with cheek retractor and avoid staining foods and beverages during entire length of treatment and 24 hours after completion of treatment.

WARNING: Keep out of reach of children. Avoid contact with eyes or skin. If contact occurs, wipe gel off and rinse with water.

DO NOT USE IF YOU: are pregnant/nursing, have poor dental health (decaying teeth, exposed roots, gum disease), wear braces, had recent oral surgery, or are allergic to peroxide. Home treatments carry minor risks, and you should consult your dentist.

RISKS: A very small percentage of people might experience an allergic type reaction to peroxide, which usually manifests as burning of the gums and/or blistering of the lips. If this occurs, discontinue use immediately, rinse the irritated area well, and consult your doctor or dentist if desired. Some people will experience minor tooth sensitivity or gum irritation. This will disappear: 1) Within a few days after discontinuing use; 2) Decreasing the length of your bleaching sessions; or 3) Increasing the amount of time between bleaching sessions; or 4) Using desensitizing gel or toothpaste.

Discontinue use if you feel a sharp pain in a tooth that lasts more than a few seconds, and consult with your dentist immediately.

Use only under the supervision of a dentist.

The manufacturer and seller accept no responsibility for any usage of this product.

INGREDIENTS: GLYCERIN, UREA PEROXIDE, AQUA, POTASSIUM NITRATE, ACRYLATES/C10-30 ALKYL ACRYLATE CROSSPOLYMER, MENTHA PIPERITA (PEPPERMINT) OIL, SODIUM HYDROXIDE, DISODIUM EDTA

CONTENTS: 2 mL 36% Carbamide Peroxide Whitening Gel

MANUFACTURED BY:
Beaming White, LLC
Vancouver, WA 98665 USA



Save 70% on Minerals™ Enamel Booster with this code: **SEVENTY**
www.whitesmilecentral.com/store